



MEDISEPT
Perfect disinfection



Velox[®] Duo Wipes

Alcoholic wipes for cleaning and rapid disinfection of small surfaces and medical equipment

Wipes for cleaning and rapid disinfection of small surfaces in medical field, nursing homes, sanatoriums, fitness and SPA centres, hairdressing, massage and beauty salons, kindergartens and school. The product is suitable for surfaces having contact with food.

Product features:

- Active in 30 secs against Bacteria, Fungi, Tuberculosis, enveloped viruses, Adeno and Rota viruses.
- Recommended for medical equipment, rehabilitation equipment, dental handpieces, patient beds, cabinets, tabletops
- For disinfection of medical devices surfaces having contact with food.
- Non-woven wipes 19x15 cm, 50 g/m²
- Dermatologically tested.

User benefits:

- Guaranteed effectiveness confirmed by tests.
- High reduction of infection risk.
- Rapid effect of clean and disinfected surface.
- Refreshing effect.
- Economical in usage.



Wipes



Active against
B, F, Tbc, V
in 30 secs



Non-invasive
medical devices



Tea tonic
scent

Velox® Duo Wipes

Application:

Wipes for cleaning and rapid disinfection of small surfaces in medical field, nursing homes, sanatoriums, fitness and SPA centres, hairdressing, massage and beauty salons, kindergartens and school. The product is suitable for surfaces having contact with food. Recommended for small surfaces of medical equipment: operating beds and chairs, beds, medical and surgical equipment and disinfection of dental tips before sterilisation process, prosthetic spoons and silicon moulds. The wipes disinfect surfaces killing in 30 seconds bacteria, fungi, mycobacteria, enveloped viruses (such as Vaccinia, BVDV, SARS-CoV-2, HIV, HBV, HCV), Rota and Noro viruses.

	Conditions		
	EN 14885	clean	dirty
Bacteria (including MRSA)	EN 13727	30 sec.	30 sec.
	EN 16615	30 sec.	30 sec.
Fungi (C. albicans)	EN 13624	15 sec.	15 sec.
	EN 16615	30 sec.	30 sec.
Mycobacteria tuberculosis (M. terrae)	EN 14348	30 sec.	30 sec.
Enveloped viruses (HIV, HBV, HCV, Influenza A, B, C, Vaccinia, SARS-Cov-2, Herpes simplex, Ebola)	EN 14476	30 sec.	60 sec.
BVDV	EN 14476	30 sec.	30 sec.
Rota virus	EN 14476	30 sec.	60 sec.
Noro virus	EN 14476	30 sec.	30 sec.

Directions for use:

Take a wipe out of the package and wipe the surface. Wait 30 seconds. In dirty conditions operating time against enveloped viruses and Rota virus is extended up to 60 seconds. In case of disinfecting surfaces having contact with food, rinse them with potable water. Other surfaces do not require rinsing with water. Close the package tightly. After opening the package, use the wipes within 21 days. Do not use if the wipe is dry. Do not use on surfaces sensitive to alcohol (e.g. acrylic glass). For professional use only.

Ingredients:

box/refill 100 pcs contain: 250 g of soaking agent.
flow pack 50 pcs contain: 173 g of soaking agent.
100 g of soaking agent contains: ethanol, propan-2-ol

Packaging units:

100 pieces box,
100 pieces refill
50 wipes in a flow pack with self-adhesive closing




Medical device class II a within the meaning of the Medical Devices Act. Intended for disinfecting non-invasive medical devices.

Permission number for biocidal products in Poland: 7905/19

The safety data sheet and the component data sheet are available at www.medisept.biz

CE 2274

 Manufacturer/Responsible entity:
MEDISEPT Sp. z o.o.
Konopnica 159 c
21-030 Motycz, Poland

phone: + 48 81 535 22 66
export@medisept.pl
www.medisept.biz

Revision: 08.2020